


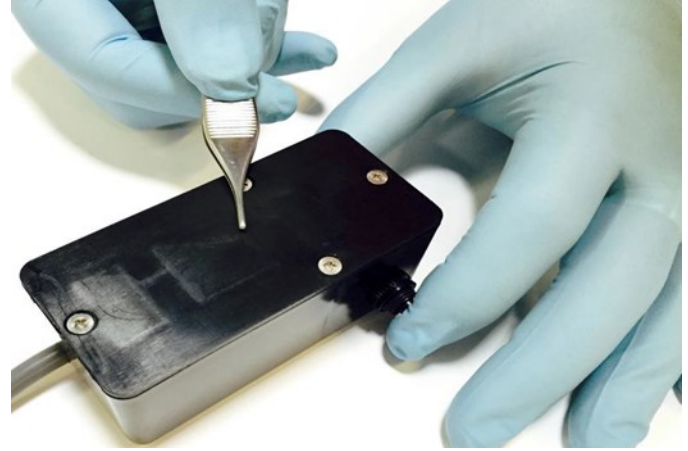

MINI-MAGIC DEMAGNETIZER

- THOROUGHLY DEMAGNETIZES STAINLESS STEEL INSTRUMENTS SIMPLY BY SLIDING THEM OVER THE SURFACE OF THE ITEM TO BE DEMAGNETIZED.
- PRESSING THE SINGLE POLE MOMENTARY SWITCH SETS UP A FIELD OF FLUX WHICH NEUTRALIZES MAGNETISM. RELEASING THE SWITCH AUTOMATICALLY SHUTS OFF THE UNIT.
- COMPLETE WITH 5 FOOT OIL RESISTANT CORD.
- SMALL COMPACT FOR MOBILE VERSATILITY
- 3 ½" L X 1 ¾" W X 1 ¼"
- SINGLE PHASE CURRENT.



CARE AND CLEANING

***DISCONNECT UNIT FROM AC SUPPLY BEFORE CLEANING AND DISINFECTING**

- IF THERE ARE ANY DEFECTS TO THE HOUSING CONTACT ELMED FOR REPAIR OR TO ORDER A REPLACEMENT.
- OBSERVE WARNINGS AND GUIDANCE ON CLEANING FLUID LABELLING REGARDING USE AND PERSONAL PROTECTIVE EQUIPMENT (PPE)
- DO NOT USE ABRASIVE CLOTHS OR CLEANERS.
- DO NOT USE AUTOMATIC WASHERS OR AUTOCLAVES
- WHEN USING DETERGENT OR DISINFECTANT WIPES ENSURE THAT EXCESS SOLUTION IS SQUEEZED FROM THE WIPE PRIOR TO USE.
- DO NOT ALLOW ANY FLUID TO ENTER THE DEVICE AND DO NOT IMMERSE IN ANY SOLUTION.
- ALWAYS WIPE OFF DISINFECTANT USING A CLOTH DAMPENED WITH CLEAN WATER.

ITEM No. **1000-01**

PORTABLE HANDHELD INSTRUMENT DEMAGNETIZER 110V

INSTRUCTIONS FOR MINI-MAGIC

PLUG INTO A STANDARD 110 VOLT AC WALL SOCKET, PRESSING THE SINGLE POLE MOMENTARY SWITCH AND SLIDING THE INSTRUMENT OVER THE SURFACE (CIRCULAR MOTION) OF THE ITEM TO BE DEMAGNETIZED. RELEASING THE SWITCH AUTOMATICALLY SHUTS OFF THE UNIT.

MINI-MAGIC IS TO BE USED AS A HAND DEMAGNETIZER ONLY AND IS NOT INTENDED FOR LONG CONTINUOUS USE. WHEN THE HOUSING GETS WARM RELEASE THE SWITCH FROM THE ON POSITION AND LET IT COOL OFF.

NOTE: IF USED IN A STERILE FIELD, THE UTILIZATION OF A STERILE SLEEVE IS REQUIRED.

WARNING: THE MANUFACTURER CAN NOT BE HELD RESPONSIBLE FOR DAMAGES OF THE DEMAGNETIZER IN ANY WAY IF THE SWITCH IS HELD IN THE ON POSITION AUTOMATICALLY. THE UNIT WILL OVERHEAT AND RESULT IN DAMAGES IF USED OVER AN EXTENDED PERIOD OF TIME.

*****Do Not Use In Bathroom, Near Water Or On Watches. Risk Of Electric Shock (Do Not Open) To Be Used Under Direct Supervision**

WE DESIGN, MANUFACTURE, & SELL THE TOOLS THE SURGEONS USE

ELMED INC.
35 N. BRANDON DR.
GLENDALE HEIGHTS, IL
60139
USA

WE SUBSCRIBE TO COST
CONTAINMENT AND
PROTECTION OF THE
ENVIRONMENT



MADE IN THE USA

THEREFORE, WE
MANUFACTURE
REUSABLE PRODUCTS
FOR A CLEANER
WORLD

PH (224) 353-6446
FAX (224) 653-8178
EMAIL
MEDICAL@ELMED.COM
WWW.ELMED.COM